**PIT & MUKERNAS  
HISFARSI 2026  
PEKANBARU-RIAU**

## **Mengenal Klasifikasi Alat Kesehatan dan Metode Sterilisasi di Central Sterile Supply Department (CSSD) Teori dan Praktik**

apt. Ammar Widitaputra, S.Farm

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
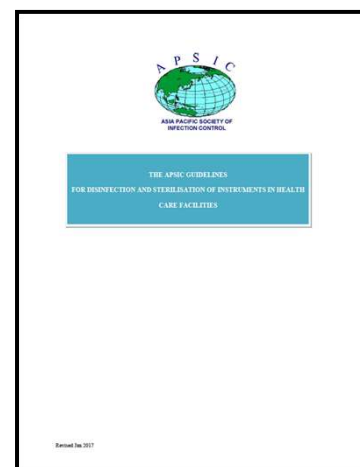
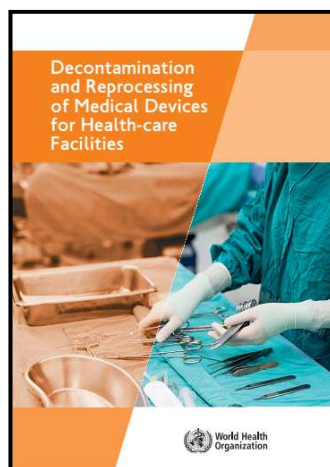
## Dekontaminasi

According to OSHA,


"the use of physical or chemical means to remove, inactivate, or destroy bloodborne pathogens on a surface or item to the point where they are no longer capable of transmitting infectious particles and the surface or item is rendered safe for handling, use, or disposal"

[29 CFR 1910.1030].

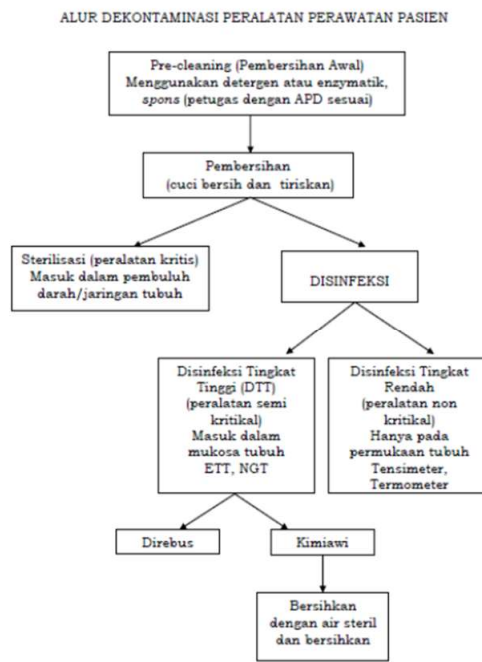In health-care facilities, the term generally refers to all pathogenic organisms.

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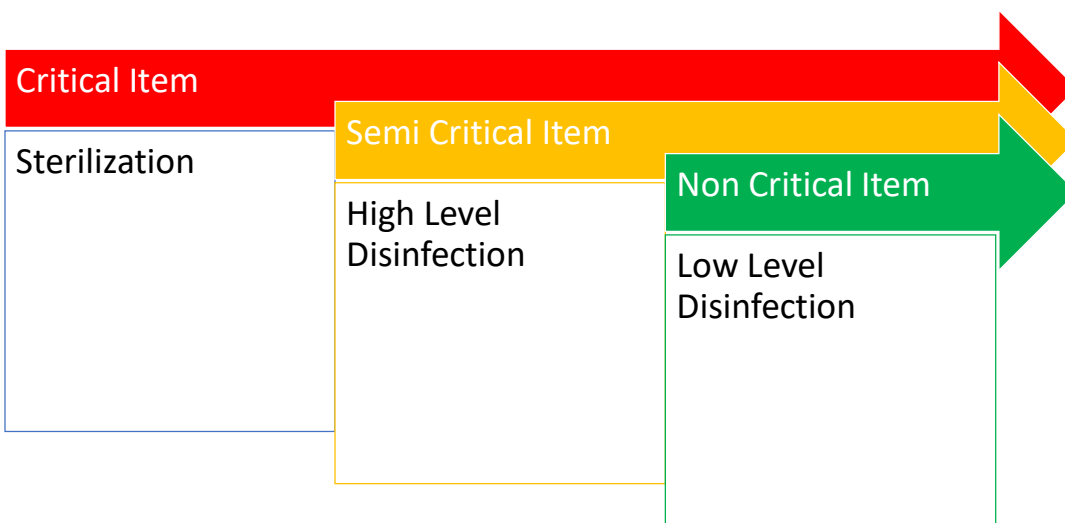
**Table 1. Level of decontamination**

<b>Cleaning</b>	The physical removal of body materials, dust or foreign material. Cleaning will reduce the number of microorganisms as well as the soil, therefore allowing better contact with the surface being disinfected or sterilized and reducing the risk of soil being fixed to the surface. Removal of soil will reduce also the risk of inactivation of a chemical disinfectant and the multiplication of microorganisms. The removal of contamination from an item to the extent necessary for further processing or for intended use. [ISO/TS 11139]
<b>Disinfection</b>	The destruction or removal of microorganisms at a level that is not harmful to health and safe to handle. This process does not necessarily include the destruction of bacterial spores.
<b>Sterilization</b>	The complete destruction or removal of microorganisms, including bacterial spores. <b>Sterility</b> State of being free from viable microorganisms. <b>Sterilization</b> Validated process used to render a product free from viable microorganisms.
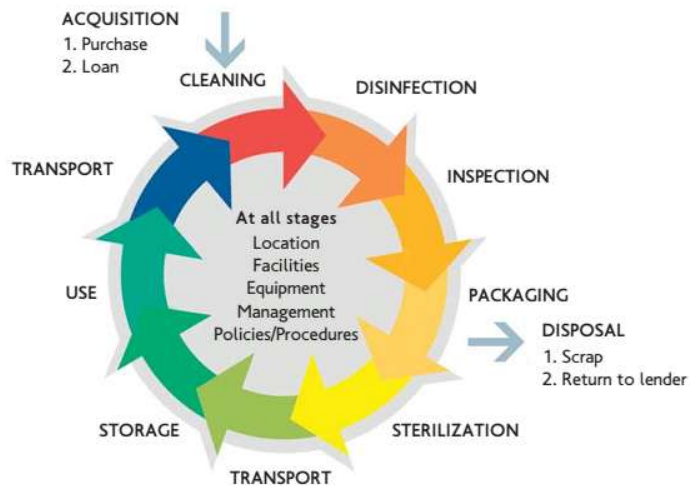
## Kegiatan Utama Dekontaminasi - CDS Cleaning, Disinfection, Sterilization

## Cleaning

Cleaning is the first and most essential step before any process of disinfection or sterilization can be carried out.

One can clean without sterilizing, but one cannot sterilize without cleaning!

## Pembersihan

Cleaning is the **removal of visible soil** (e.g., organic and inorganic material) from objects and surfaces and normally is accomplished manually or mechanically using water with detergents or enzymatic products.

Cleaning is accomplished by manual cleaning with **cleaning chemicals (detergent)** and water, **brushing or flushing**, or by using **ultrasonic and or washer disinfectors** to remove foreign material

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Water	Universal solvent
Bases, Acids, Enzymes	Hydrolysis- in the presence of water
Surfactants	Wet, emulsify, disperse & suspend solids
Chelants, Sequestering Agents	Tie up hard water & metal ions
Builders	Assist in detergency
Corrosion Inhibitors	Protect surfaces against corrosion
Biocides	Preservative, antimicrobial agents
Solvents	Solubilise

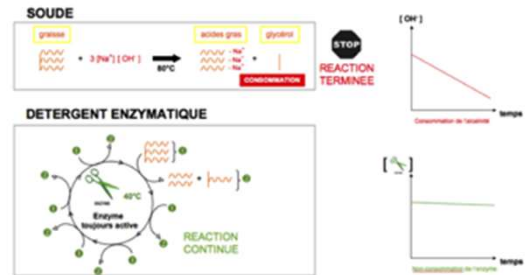
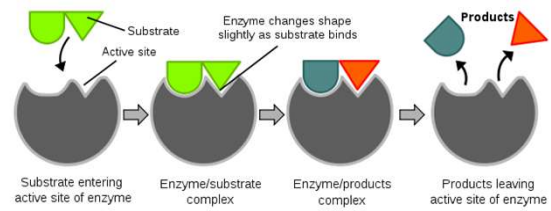
### REMEMBER

Only use appropriate detergents for instrument cleaning in the SSD. Detergents used for home cleaning or laundry use are not suitable for the cleaning of medical devices or instruments.

## Enzim

- Protease → Darah; Lipase → Lemak; Amilase → Polisakarida

- Break down larger molecules into smaller molecules (Hydrolysis)

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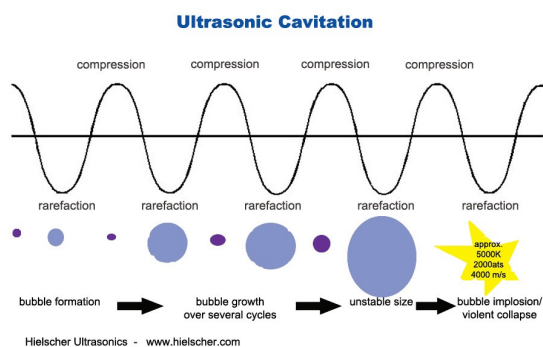
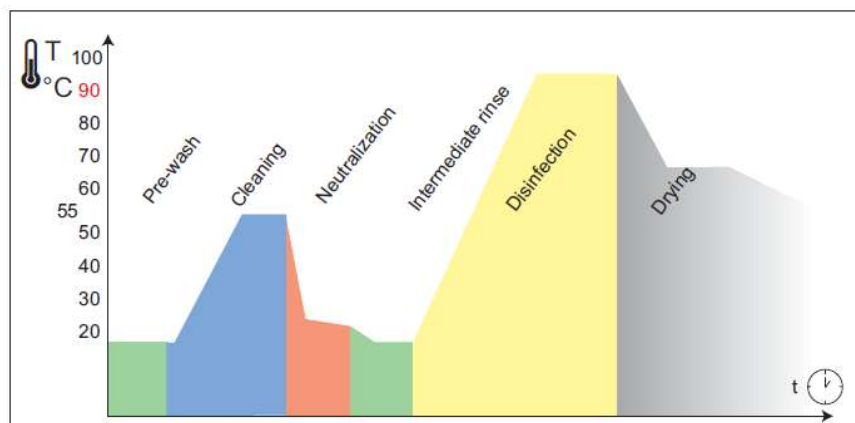
## Washer Disinfector Thermal Disinfection

Cleaning programme with thermal disinfection

## A<sub>0</sub>-Concept

Thermal disinfection with moist heat, based on the A<sub>0</sub> concept (EN ISO 15883-1), is the most common method for disinfection of medical devices in the hospital setting. A<sub>0</sub> is a physical parameter denoting the inactivation of microorganisms. The concept of A<sub>0</sub> is intended to allow equivalent disinfection efficiencies to a reference time/temperature to occur at other disinfection temperatures.

$$A_0 = \sum 10^{(T-80)/z} \Delta t$$

t = preset time interval, expressed in °C

T = Temperature in the load, in °C

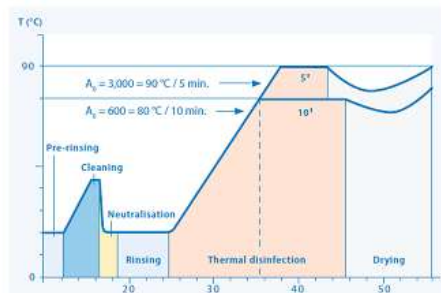
z = germ-specific temperature value

A<sub>0</sub>-Value = A-Value, if z = 10°C

## A<sub>0</sub>-Concept

Process temp. (°C)	Holding time for A <sub>n</sub> = 6000		Holding time for A <sub>n</sub> = 3000		Holding time for A <sub>n</sub> = 600		Holding time for A <sub>n</sub> = 60	
	sec	min	sec	min	sec	min	sec	min
70	60 000	1 000	30000	500,0	6000	100,0	600	10,0
80	6 000	100	3000	50,0	600	10,0	60	1,0
90	600	10	300	5,0	60	1,0	6	0,1
93	300	5	150	2,5	30	0,5	3	0,1

### Thermal disinfection in WD


# Validasi Proses Pencucian - Visual Control

**Table 4. A summary of decontamination processes and measurements for validation and their application**

PROCESS	WHAT IS MEASURED AND WHEN?	
Cleaning	Daily Use of detergent and disinfectant	Per item Cleaning results by visual control or by using a cleaning test

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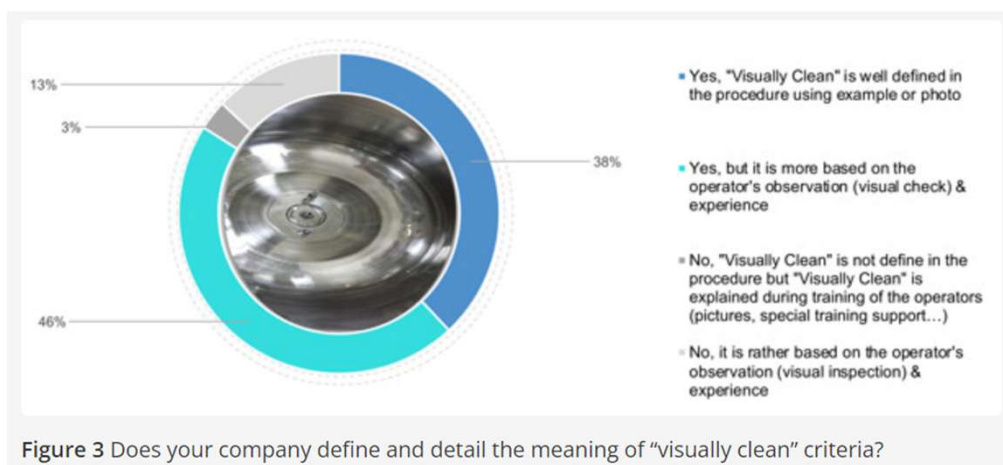## New packaging (1)

Not complying with EN or ISO standards

These boxes  
are no longer  
used, and it is  
easy to  
understand  
why...

## New packaging (2) Complying with EN or ISO standards

Containers  
with  
permanent  
filters

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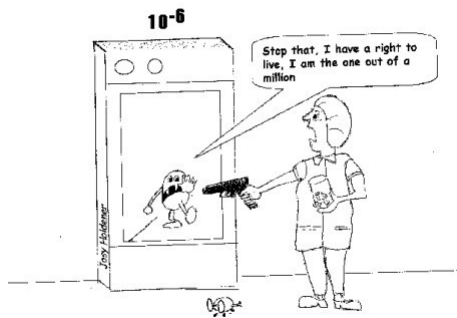
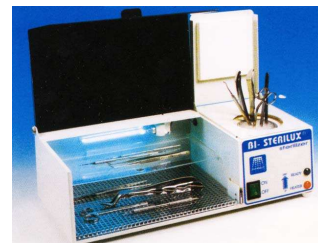
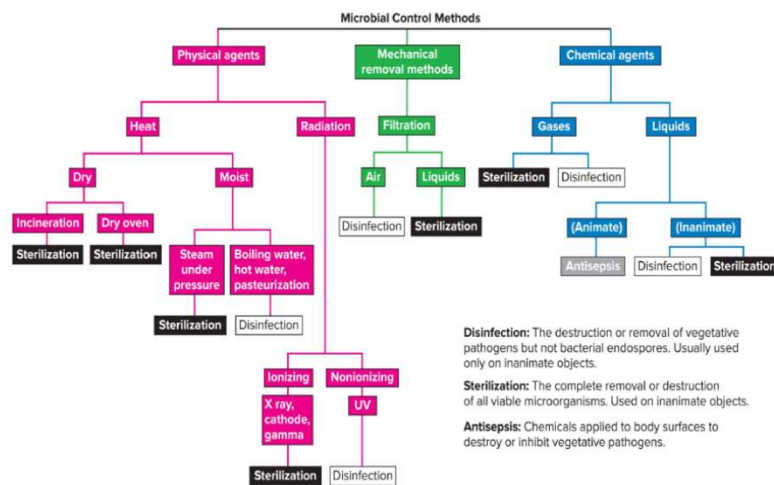
APUSIC, Asia Pacific Guidelines for Disinfection & Sterilization

## An overview of microbial control methods

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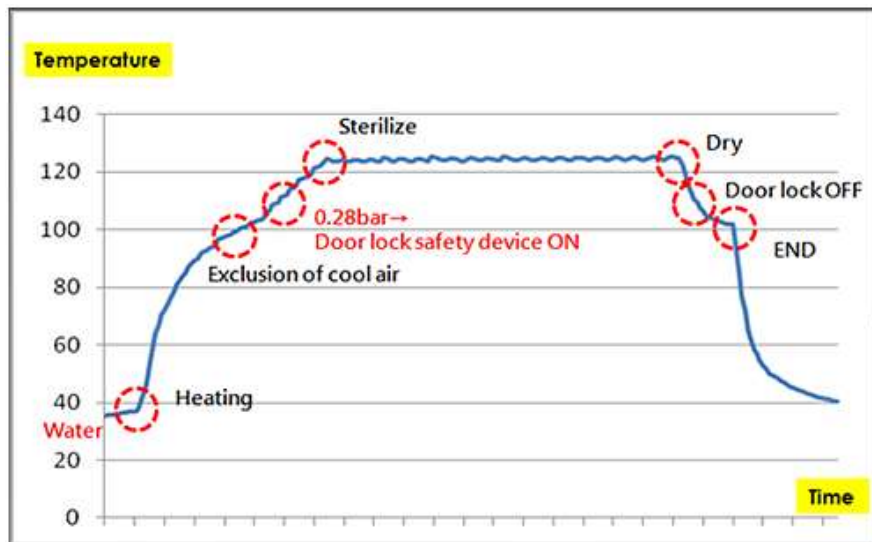
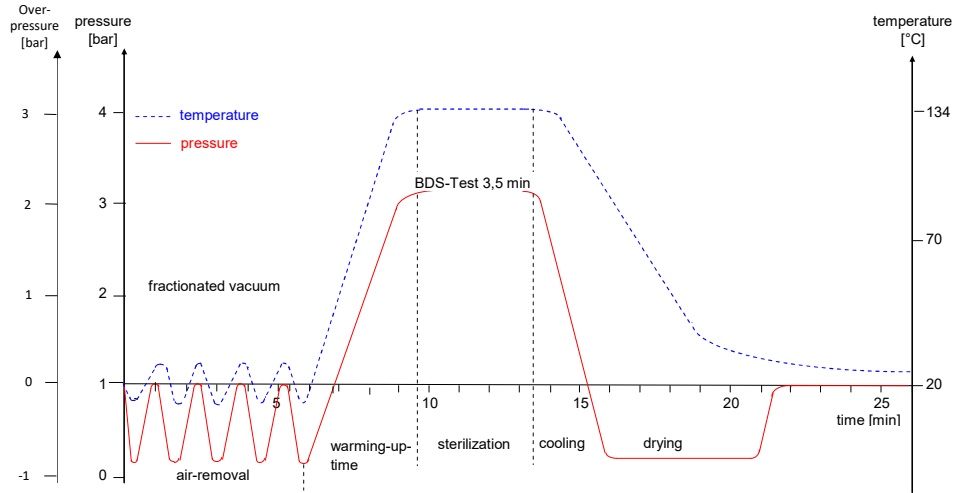
### Sub-atmospheric Fractionated Vacuum Sterilization Process

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
Table 12. Types of endoscopic procedures

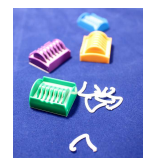Types of endoscopes	Rigid endoscope example	Flexible endoscope example	Level of decontamination
<b>Invasive</b> – passed into normally sterile body cavities or introduced into the body through a break in the skin or mucous membrane	Arthroscope Laparoscope Cystoscope	Nephroscope Angioscope Choledochoscope	Sterilization by steam or a low temperature method e.g. gas plasma
<b>Non-invasive</b> in contact with intact mucous membrane, but does not enter sterile cavities	Bronchoscope	Gastroscope Colonoscope Bronchoscope	High-level disinfection, e.g. immersion in glutaraldehyde, peracetic acid, chlorine dioxide

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### Low Temperature Steam Formaldehyde (LTSF)

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

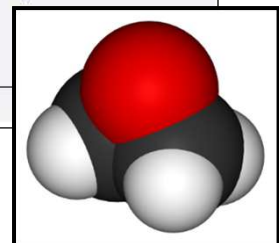
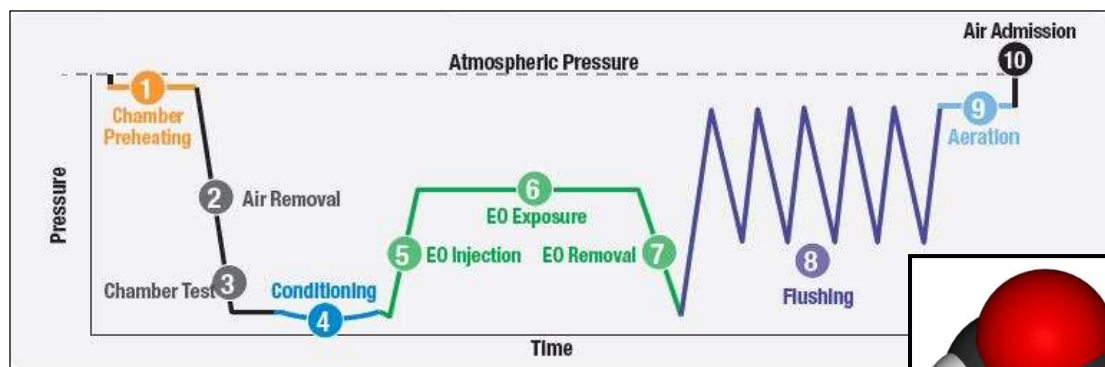
## Etilen Oksida (ETO)

# Low Temperature Steam Formaldehyde (LTSF)


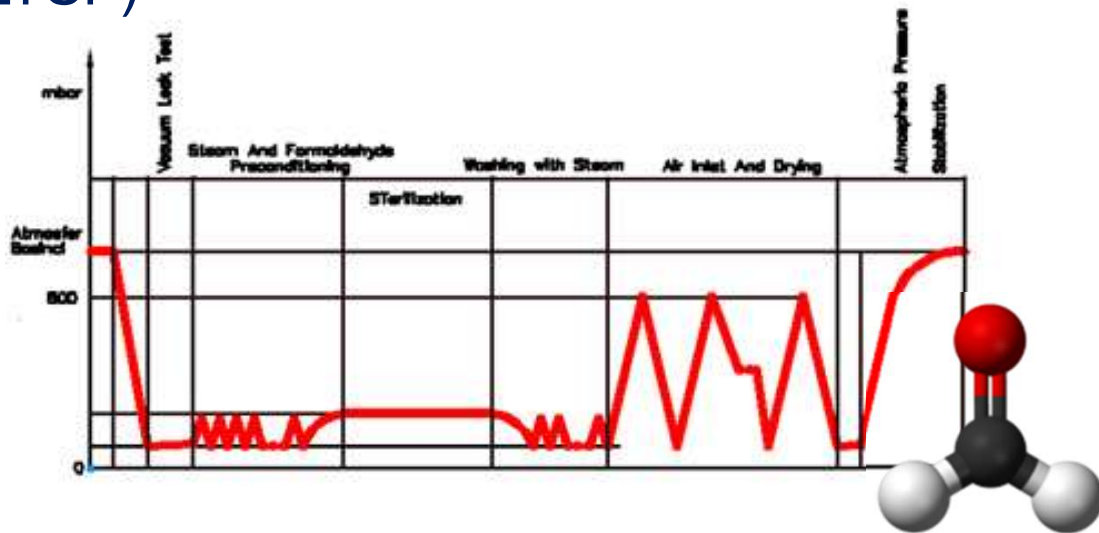
# Hidrogen Peroksida (Plasma)

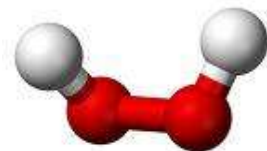**H2O2 Vapour**

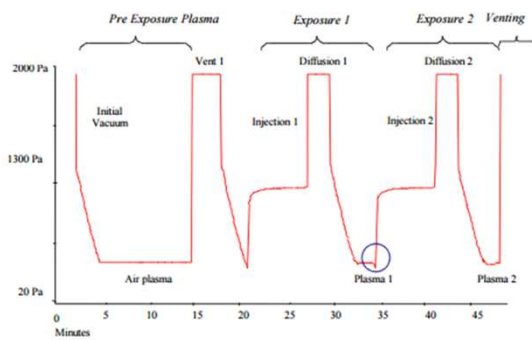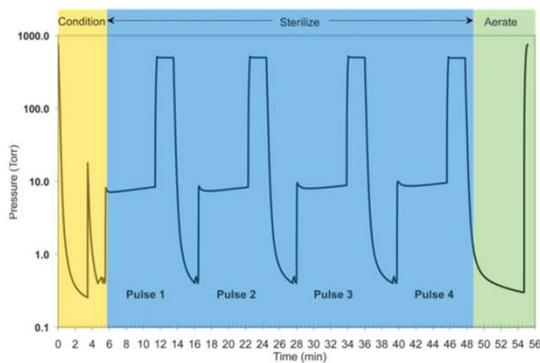**H2O2 Plasma**


## Sterilisation Process Monitoring

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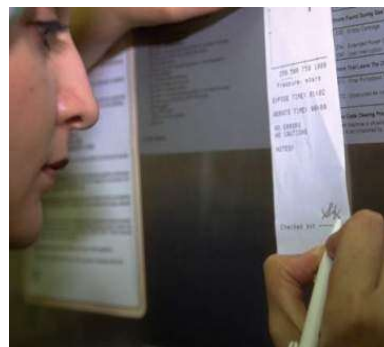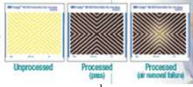
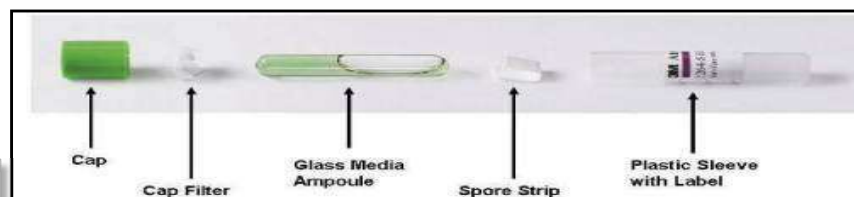
*If spores viable – process failure*

## Perkembangan BI

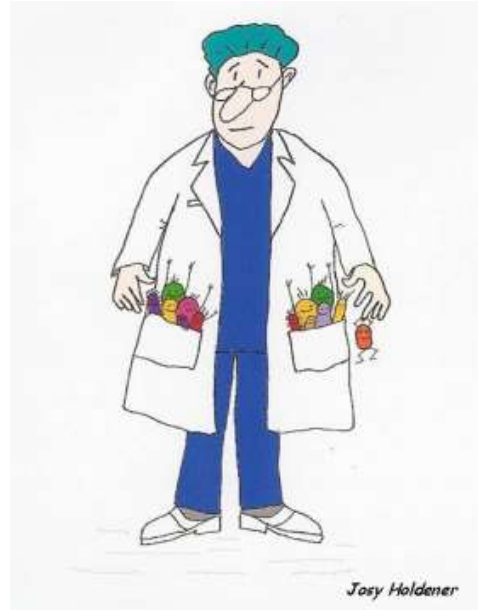
**Table 4. A summary of decontamination processes and measurements for validation and their application**

PROCESS	WHAT IS MEASURED AND WHEN?	
Cleaning	Daily Use of detergent and disinfectant	Per item Cleaning results by visual control or by using a cleaning test
Disinfection	Daily Use of disinfectant by concentration, temperature and pH of disinfectant	Per load Time of exposure
Chemical sterilizers		Per process Biological indicator Chemical indicators Physical indicator Per item External indicators
Moist heat (steam sterilizers)	Daily Bowie-Dick test for steam penetration in porous loads (pre- vacuum autoclave) (Helix test for hollow lumen instruments, if available.) Clean the chamber every week	Per process Biological indicator Chemical indicators Physical parameters met as per PQ Per item External indicators

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## WFHSS Honorary President - Wim Renders

### Curriculum vitae Wim Renders

Obtained the degree of pharmacist in 1973 at the Rijksuniversiteit Ghent, Belgium.

From 1973 to 1977 pharmacist in a private pharmacy.

From May 1977 to 2011 pharmacist in the AZ St Jan hospital, Bruges, Belgium.

- Responsible for the Central Sterilization Department from 29/05/1991
- Adjunct-head pharmacist from 01/05/1993
- Head pharmacist a.i. from 09/02/2009 to 2011

From 1996 to 2010 president of the Flemish Society for Sterilization.

From 1999 to 2014 president of the World Forum for Hospital Sterile Supply (WFHSS, [www.wfhss.com](http://www.wfhss.com)).

Editor of "Zentral Sterilisation" from 1999 to 2015.

Currently the Honorary-President of WFHSS.

# WFHSS Education Officer – Carolina Chiodini

**Pharmacist Specializing in Sterilization:  
CAROLINA CHIODINI**

**EDUCATION**

Licenciatura in Pharmacy – School of Chemistry National University of Cordoba – ( 1998)	Argentina (1998-2005)
Specializing in Sterilization and biomedical devices - Juan Agustín Maza University – Mendoza – September (2008)	Argentina (2005-2018)
Certification of the Certification Board for Sterile Processing and Distribution of the United States of America (CBSPD), (2010)	France (2006-2015).
Interuniversity Certificate of Advanced Studies in Hospital Sterilization. Joseph Fourier University – Grenoble, Claude Bernard University – Lyon. ( 2012 )	Argentina (2006-2012)
Implementation of Quality Processes with guidelines towards the National Quality Award in the Sterilization Center at the Dr. Natalio Burd Hospital – Centenario, Argentina (2007).	Argentina (2008-2018)
Representative of the Undersecretary of Health in the first Quality Forum in the Province of Neuquén about Application of Criteria of the National Quality Award for the Public Sector (2009).	
Internships at 19 Sterilization Centers in French, Swiss and Spanish Public Hospitals (2011 - 2012).	
Lecturer at the 16 <sup>th</sup> World Sterilization Congress of the WFHSS in Lille, France on October 9 <sup>th</sup> , ( 2015 ) “Training of Pharmaceutical professionals specialists and technicians in sterilization in Argentina”	

*CC*

# WFHSS last President – Christine Denis

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© Sylvia Garcia-Houchins